



Thyroid Transcription Factor-1 (TTF-1) Monoclonal Mouse Anti-Thyroid Transcription Factor-1 Clone SPT24

REF 014-1028

Ready-To-Use ■ 100 Tests / 50 Tests

Concentrate ■ 1ml

INTENDED USE

IVD For in vitro diagnostic use.

Celerus monoclonal mouse anti-thyroid transcription factor-1, clone SPT24, is intended for laboratory use to identify thyroid and lung tumors using light microscopy. It may be used with frozen tissue or with formalin-fixed paraffin-embedded tissue.

Positive results aid in the classification of normal and abnormal cells/tissues and serve as an adjunct to conventional histopathology. The clinical interpretation of any positive staining or its absence should be complemented by morphological and histological studies with proper controls. Evaluations should be made by a qualified individual in conjunction with the patient's clinical history and other diagnostic test results.

Refer to the Wave Instrument Operator's Manual for additional information concerning Materials Required but Not Provided; Storage; Staining Procedure; Troubleshooting; Interpretation of Staining; and General Limitations.

SUMMARY AND EXPLANATION

Thyroid transcription factor-1 is expressed in normal adult lung and thyroid and has been reported in some carcinomas of these tissues. Thyroid transcription factor-1 antibody can be used as part of a panel to differentiate lung and thyroid primary cancers from other carcinomas that metastasize to the lung, such as breast and colon carcinomas. It specifically binds to antigens in the lung and thyroid epithelia.

PRINCIPLE OF PROCEDURE

Immunohistochemistry is a multi-step process to identify specific cell markers within tissue biopsies or tumor specimens. The sequential steps include antigen retrieval (optional), antibody application, and antibody visualization followed by optional counterstaining. Specimens are then coverslipped and observed under light microscopy by trained personnel. Normally, multiple antibodies are tested to determine lineage and cell cycle markers. The Celerus Wave is an automated instrument that performs immunohistochemistry stains. For further information on the staining procedure, refer to the Celerus Wave Operator's Manual.

MATERIALS AND METHODS

Reagent Provided

Clone
SPT24

Ig Class

IgG 1, kappa

Immunogen

Prokaryotic recombinant protein corresponding to a 123 amino acid fragment of the N-terminal region of the thyroid transcription factor-1 molecule.

Ready-To-Use in Primary Antibody Cartridge

Celerus anti-thyroid transcription factor-1 is provided with ProClin 300 as a preservative, in a Primary Antibody Cartridge (PAC), a self-contained dispenser of reagents. Each PAC contains sufficient reagent to complete 100 stained slides. PACs must remain upright to avoid spilling. PAC must be primed before first use. See Celerus Wave Operator's Manual for details.

Concentrated Antibody

Liquid

Liquid concentrated antibody is provided containing 15 mM sodium azide as a preservative and 1% bovine serum albumin as a carrier protein.

Lyophilized

Lyophilized antibody is provided containing 15mM sodium azide as a preservative. Reconstitute vial with 1.0 ml distilled water.

Dilution

The suggested dilution is 1:25– 1:50. This is a guide only and users should determine their own optimal working dilutions.

SPECIFICITY

Human thyroid transcription factor-1.

MATERIALS REQUIRED BUT NOT PROVIDED

- Wave instrument
- Wave slide rack
- Positively-charged microscope slides, appropriately labeled
- Timer
- Celerus Riptide for antigen retrieval (or equivalent)
- Celerus Target Retrieval Solution (or equivalent)
- Slide drying chamber
- Xylene or xylene substitute
- Reagent alcohol or ethyl alcohol
- Distilled or deionized water
- TBS wash buffer, pH 7.6
- Positive and negative tissue controls
- Celerus Negative Control Reagent (or equivalent)
- Mounting Medium
- Cover slips

STORAGE AND HANDLING

Ready-to-Use PAC, Liquid Concentrated and Lyophilized Antibody

Store reagent at 2–8 °C. Do not freeze. The reagent is stable until the expiration date on the container. Do not use reagent after the expiration date, as the activity cannot be ensured.

Reconstituted Antibody

For reconstituted antibody, the reagent is stable for at least two months when stored at 4 °C. For long-term storage it is recommended that aliquots of the antibody be stored at -20 °C. Repeated freezing and thawing of the antibody should be avoided.

There are no signs to indicate instability of this reagent. To ensure a valid staining assay, the use of positive and negative tissue controls is recommended. Contact your Celerus representative if there are stability concerns prior to the expiration date.

PRECAUTIONS

- For professional users.
- Minimize microbial contamination of reagents or an increase in nonspecific staining may occur.
- As with any product derived from biological sources, proper handling procedures should be used.
- A Material Safety Data Sheet is available for professional users on request.

- ProClin 300 is classified per applicable European Community (EC) Directives as: Irritant (Xi). The following are the appropriate Risk (R) and Safety (S) phrases.



R36 Irritating to eyes

R43 May cause sensitization by skin contact

S24 Avoid contact with skin

S26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice

S35 This material and its container must be disposed of in a safe way

S37/39 Wear suitable gloves and eye/face protection

S46 If swallowed, seek medical advice immediately and show this container or label.

WASTE DISPOSAL

Adhere to all local laws when disposing of the PAC.

PACKAGING DAMAGE

DO NOT USE a PAC if it is leaking, has leaked, has spilled, cannot be primed, or has visually apparent physical damage.

SPECIMEN COLLECTION AND HANDLING

Formalin-fixed paraffin embedded (FFPE) tissues, frozen tissues, or smears are suitable for use. Wave detection kits have been optimized for tissues fixed with 10% formalin. Ideally, each 4–6µ tissue section should be placed on charged slides on the lower 2/3 of the slide. Very large sections should be placed 1/4 inch below the lower end of the slide label.

Slides should be baked overnight at 37 °C, or at 60 °C for one hour.

Use standard histochemical techniques to deparaffinize processed slides. For uniformity of staining results, it is recommended that target retrieval be performed using the Celerus Riptide and Celerus Target Retrieval Solution (or equivalent) at 112 °C for 5 minutes. Avoid drying of the tissue specimen during this process. After all slides to be stained have been inserted and reagents mounted on the instrument, start the staining run.

When the slides have completed the staining run, remove them from the instrument, coverslip, and view under light microscopy.

PRODUCT-SPECIFIC LIMITATIONS

Thyroid transcription factor-1 antibody, when used on the Wave instrument, detects antigens that survive routine formalin fixation, tissue processing, and sectioning. Users who deviate from recommended test procedures are responsible for interpretation and validation of patient results.

RESULTS EXPECTED/ PERFORMANCE CHARACTERISTICS

Normal Tissues

Clone SPT24 detects thyroid transcription factor-1 protein in the nucleus of follicular epithelial cells of the thyroid, and type II pneumocytes and Clara cells of the lung.

Abnormal Tissues

Clone SPT24 stained 11/13 adenocarcinomas of the lung, 5/7 small cell carcinomas of the lung, 1/2 thyroid papillary adenocarcinomas, 2/2 medullary carcinomas of the thyroid, 1/1 thyroid adenomas, and 1/2 Hashimoto's thyroiditis. It did not stain squamous cell carcinomas of the lung (0/11) or a variety of other carcinomas (0/53) including carcinomas of the colon, stomach, pancreas, breast, liver, kidney, prostate and ovary, melanomas, lymphomas, sarcomas, seminomas, mesotheliomas, and carcinoids.

Thyroid transcription factor-1 antibody SPT24 is recommended for use in the classification of tumors of the thyroid and lung.

REFERENCES

- Johansson L. Histopathologic classification of lung cancer: relevance of cytokeratin and TTF-1 immunophenotyping. *Ann Diagn Pathol.* 8(5):259-267 (2004).
- Bejarano PA, Nikiforov YE, Swenson ES, Biddinger PW. Thyroid transcription factor-1, thyroglobulin, cytokeratin 20 and cytokeratin 12 in thyroid neoplasms. *Appl Immunohistochem Mol Morphol.* 8(3):189-194 (2000).
- Khoor A, Whitsett JA, Stahlman MT, Olson SJ, Cagle PT. Utility of surfactant protein B precursor and thyroid transcription factor-1 in differentiating adenocarcinoma of the lung from malignant mesothelioma. *Hum Pathol.* 30(6):695-700 (1999).
- Bingle CD. Thyroid transcription factor-1. *Int J Biochem Cell Bio.* 29(12):1471-1473 (1997).
- Folpe AL, Gown AM, Lamps LW, Garcia R, Dail DH, Zarbo RJ, Schmidt RA. Thyroid transcription factor-1: Immunohistochemical evaluation in pulmonary neuroendocrine tumors. *Mod Pathol.* 12(1):5-8 (1999).
- Stenhouse G, Fyfe N, King G, Chapman A, Kerr KM. Thyroid transcription factor-1 in pulmonary adenocarcinoma. *J Clin Pathol.* 57:383-387 (2004).
- Abutally AS, Addis BJ, Roche WR. Immunohistochemistry in the distinction between malignant mesothelioma and pulmonary adenocarcinoma: a critical evaluation of new antibodies. *J Clin Pathol.* 55:662-668 (2002).
- Ordóñez NG. The diagnostic utility of immunohistochemistry in distinguishing between epithelioid mesotheliomas and squamous carcinomas of the lung: a comparative study. *Mod Pathol.* 19(3):417-428 (2006).



Vector Laboratories, Ltd.
3 Accent Park, Bakewell Road
Orton Southgate, Peterborough
PE2 6XS, England



Celerus Diagnostics
1005 Mark Avenue
Carpinteria, CA 93013 USA

TECHNICAL SUPPORT 888-444-3613
CUSTOMER SERVICE 888-444-8918

www.celerusdiagnostics.com