



Mammaglobin

Monoclonal Mouse Anti-Mammaglobin

Clone 304-1A5

REF 014-1124

Ready-To-Use ■ 100 Tests / 50 Tests

Concentrate ■ 1mL

INTENDED USE

IVD For in vitro diagnostic use.

Celerus monoclonal mouse anti-Mammaglobin, clone 304-1A5, is intended for laboratory use in identifying mammaglobin antigen in normal or neoplastic tissue using light microscopy. Mammaglobin is present in nearly all types of breast adenocarcinoma regardless of tumor differentiation and type. Adenocarcinomas from other organs rarely express mammaglobin. This antibody may be used with frozen or formalin-fixed paraffin-embedded tissues.

Positive results aid in the classification of normal and abnormal cells/tissues and serve as an adjunct to conventional histopathology. The clinical interpretation of any positive staining or its absence should be complemented by morphological and histological studies with proper controls. Evaluations should be made by a qualified individual in conjunction with the patient's clinical history and other diagnostic test results.

Refer to the Wave Instrument Operator's Manual for additional information concerning Materials Required but Not Provided; Storage; Staining Procedure; Troubleshooting; Interpretation of Staining; and General Limitations.

SUMMARY AND EXPLANATION

Mammaglobin is a breast-associated glycoprotein distantly related to secretoglobulin family that includes human uteroglobin and lipophilin. Unlike other secretoglobulin family members, mammaglobin mRNA expression is breast specific, which has been shown to be a sensitive marker of occult breast cancer cells in sentinel lymph nodes and peripheral blood. Mammaglobin, in conjunction with a panel of other markers, may aid in the identification of breast carcinomas presenting at metastatic sites.

PRINCIPLE OF PROCEDURE

Immunohistochemistry is a multi-step process to identify specific cell markers within tissue biopsies or tumor specimens. The sequential steps include antigen retrieval (optional), antibody application, and antibody visualization followed by optional counterstaining. Specimens are then coverslipped and observed under light microscopy by trained personnel. Normally, multiple antibodies are tested to determine lineage and cell cycle markers. The Celerus Wave is an automated instrument that performs immunohistochemistry stains. For further information on the staining procedure, refer to the Celerus Wave Operator's Manual.

MATERIALS AND METHODS

Reagent Provided

Clone

304-1A5

Ig Class

IgG₁, kappa

Immunogen

Recombinant mammaglobin protein (containing amino acids 15–93).

Ready-To-Use in Primary Antibody Cartridge

Celerus anti-Mammaglobin is provided with ProClin 300 as a preservative, in a Primary Antibody Cartridge (PAC), a self-contained dispenser of reagents. Each PAC contains sufficient reagent to complete 50 or 100 stained slides. PACs must remain upright to avoid spilling. PAC must be primed before first use. See Celerus Wave Operator's Manual for details.

Concentrated Antibody

Liquid

Liquid concentrated antibody is provided containing 15 mM sodium azide as a preservative and 1% bovine serum albumin as a carrier protein.

Lyophilized

Lyophilized antibody is provided containing 15mM sodium azide as a preservative. Reconstitute vial with 1.0 ml distilled water.

Dilution

The suggested dilution is 1:50–1:100. This is a guide only and users should determine their own optimal working dilutions.

SPECIFICITY

Human, monkey, mouse, and rabbit mammaglobin.

MATERIALS REQUIRED BUT NOT PROVIDED

- Wave instrument
- Wave slide rack
- Positively-charged microscope slides, appropriately labeled
- Timer
- Celerus Riptide for antigen retrieval (or equivalent)
- Celerus Target Retrieval Solution (or equivalent)
- Slide drying chamber
- Xylene or xylene substitute
- Reagent alcohol or ethyl alcohol
- Distilled or deionized water
- TBS wash buffer, pH 7.6
- Positive and negative tissue controls
- Celerus Negative Control Reagent (or equivalent)
- Mounting Medium
- Cover slips

STORAGE AND HANDLING

Ready-to-Use PAC, Liquid Concentrated and Lyophilized Antibody

Store reagent at 2–8 °C. Do not freeze. The reagent is stable until the expiration date on the container. Do not use reagent after the expiration date, as the activity cannot be ensured.

Reconstituted Antibody

For reconstituted antibody, the reagent is stable for at least two months when stored at 4 °C. For long-term storage it is recommended that aliquots of the antibody be stored at –20 °C. Repeated freezing and thawing of the antibody should be avoided.

There are no signs to indicate instability of this reagent. To ensure a valid staining assay, the use of positive and negative tissue controls is recommended. Contact your Celerus representative if there are stability concerns prior to the expiration date.

PRECAUTIONS

- For professional users.
- Minimize microbial contamination of reagents or an increase in nonspecific staining may occur.
- As with any product derived from biological sources, proper handling procedures should be used.
- A Material Safety Data Sheet is available for professional users on request.
- ProClin 300 is classified per applicable European Community (EC) Directives as: Irritant (Xi). The following are the appropriate Risk (R) and Safety (S) phrases.



R36 Irritating to eyes

R43 May cause sensitization by skin contact

S24 Avoid contact with skin

S26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice

S35 This material and its container must be disposed of in a safe way

S37/39 Wear suitable gloves and eye/face protection

S46 If swallowed, seek medical advice immediately and show this container or label.

WASTE DISPOSAL

Adhere to all local laws when disposing of the PAC.

PACKAGING DAMAGE

DO NOT USE a PAC if it is leaking, has leaked, has spilled, cannot be primed, or has visually apparent physical damage.

SPECIMEN COLLECTION AND HANDLING

Formalin-fixed paraffin embedded (FFPE) tissues, frozen tissues, or smears are suitable for use. Wave detection kits have been optimized for tissues fixed with 10% formalin. Ideally, each 4–6µ tissue section should be placed on charged slides on the lower 2/3 of the slide. Very large sections should be placed 1/4 inch below the lower end of the slide label.

Slides should be baked overnight at 37 °C, or at 60 °C for one hour.

Use standard histochemical techniques to deparaffinize processed slides. For uniformity of staining results, it is recommended that target retrieval be performed using the Celerus Riptide and Celerus Target Retrieval Solution (or equivalent) at 112 °C for 5 minutes. Avoid drying of the tissue specimen during this process. After all slides to be stained have been inserted and reagents mounted on the instrument, start the staining run.

When the slides have completed the staining run, remove them from the instrument, coverslip, and view under light microscopy.

PRODUCT-SPECIFIC LIMITATIONS

Mammaglobin antibody, when used on the Wave instrument, detects antigens that survive routine formalin fixation, tissue processing, and sectioning. Users who deviate from recommended test procedures are responsible for interpretation and validation of patient results.

RESULTS EXPECTED / PERFORMANCE CHARACTERISTICS

Normal Tissues

This antibody labels ductal and lobular epithelial cells in normal breast tissue.

Abnormal Tissues

In tumor cells, this antibody is reactive with all types of breast adenocarcinoma regardless tumor differentiation and types. Adenocarcinomas from other organs rarely express mammaglobin.

REFERENCES

Watson MA, Dintzis S, Darrow CM, Voss LE, DiPersio J, Jensen R, Fleming TP. Mammaglobin expression in primary, metastatic, and occult breast cancer. *Cancer Res* 1999; 59:3028–3031.

Min CJ, Tafra L, Verbanck KM. Identification of superior markers for polymerase chain reaction detection of breast cancer metastases in sentinel lymph nodes. *Cancer Res* 1998; 58(20):4581–4584.

Grünewald K, Haun M, Fiegl M, Urbanek M, Müller-Holzner E, Massoner A, Riha K, Probst A, Marth C, Gastl G. Mammaglobin expression in gynecologic malignancies and malignant effusions detected by nested reverse transcriptase-polymerase chain reaction. *Laboratory Investigation* 2002; 82(9):1147–1153.

Fanger GR, Houghton RL, Retter MW, Hendrickson RC, Babcock J, Dillon DC, Durham MD, Reynolds LD, Johnson JC, Carter D, Fleming TP, Roche PC, Persing DH, Reed SG. Detection of mammaglobin in the sera of patients with breast cancer. *Tumour Biol* 2002; 23(4):212–221.

Ciampa A, Fanger G, Khan A, Rock KL, Xu B. Mammaglobin and CRxA-01 in pleural effusion cytology: potential utility of distinguishing metastatic breast carcinomas from other cytokeratin 7-positive/cytokeratin 20-negative carcinomas. *Cancer* 2004; 102(6):368–372.

Stachurski D, Fanger G, Xu B, Edminston K, Rock KL, Khan A. CRxA-01 and mammaglobin expression in breast cancer: potential role in differentiating breast cancer from other CK7+ /CK20- tumors. (205) *Mod Path* 2003; 16(1):48A.



Vector Laboratories, Ltd.
3 Accent Park, Bakewell Road
Orton Southgate, Peterborough
PE2 6XS, England

Celerus Diagnostics
1005 Mark Avenue
Carpinteria, CA 93013 USA

TECHNICAL SUPPORT 888-444-3613
CUSTOMER SERVICE 888-444-8918

www.celerusdiagnostics.com