

WICKING CASSETTE

REF 014-2500-000

INSTRUCTIONS FOR USE

PAGE 1 of 1

INTENDED USE

For in vitro diagnostic use.

The Wicking Cassette is intended for laboratory use to remove reagents from microscope slides being processed by the Celerus Wave® and Celerus Wave® RPD Systems. The Wicking Cassette is intended for use exclusively with the Wave and Wave RPD Systems. The test capacity of a Wicking Cassette varies depending on the staining protocol being performed.

The Wave or Wave RPD System will indicate when the Wicking Cassette does not have enough capacity remaining to perform a particular test.

INSTRUCTIONS FOR USE

Refer to the Celerus Wave® or Celerus Wave® RPD Operator's Manual.

MATERIALS REQUIRED BUT NOT PROVIDED

Refer to the Celerus Wave® or Celerus Wave® RPD Operator's Manual.

STORAGE AND HANDLING

Refer to the Celerus Wave® or Celerus Wave® RPD Operator's Manual.

WASTE DISPOSAL

1. Adhere to all local laws regarding hazardous waste when disposing of the Wicking Cassette.
2. If used with fresh or frozen tissue the cassette is considered a biohazard.
3. A biohazard bag and label are included in the package for your convenience.









PACKAGING DAMAGE

DO NOT USE a Wicking Cassette if it is cracked, leaking, or has other visually apparent physical damage.

PRECAUTIONS

1. For professional users.
2. If the Wicking Cassette is used with frozen human tissue, it should be considered a biohazard and the supplied biohazard label should be applied to the Wicking Cassette.
3. Users should wear gloves when handling the Wicking Cassette.
4. During a run, the Wicking Cassette absorbs hazardous chemicals contained within the PACs and LRMs. These remain in the Wicking Cassette after a run is completed. See INSTRUCTIONS FOR USE for the PACs and/or LRMs for details on the hazards posed by the chemicals.
5. Refer to the Celerus Wave® or Celerus Wave® RPD Operator's Manual for additional precautions.

EXPLANATION OF SYMBOLS

REF Catalog Number	 Temperature Limitations	LOT Batch Code	 Sufficient for <n> Tests	 Harmful, Irritant	 Warning
IVD In Vitro Diagnostic Medical Device	 Use By	 European Conformity	EC REP EC Representative	 Manufacturer	 Refer to Instructions for Use

